

Sample Collection Set

The sample collection set is used for storage and transportation of human specimen. It offers biological buffer, maintains normal PH value of organism and stability of nucleic acid and prevents nucleic acid degradation. Besides, the product can inactivate the advanced structure of pathogen protein, deprive biological activity and reduce infectivity and pathogenicity.



Stable

prevent nucleic acid degradation



Safe

inactivated, reduce infectivity and pathogenicity


Parameters

Specification	48T/kit		
Product Code	T326H	T326HB	T326HC
Specimen Collection Method	Oropharyngeal swab	Nasopharyngeal Swab	Nasopharyngeal swab & Oropharyngeal swab
Storage and Stability	Store under room temperature; Validity period:12 months		
Usage	Storage and transportation of human specimen		


Test Procedure




① Swab on the posterior wall of the pharynx tonsil recess, lateral wall, etc. Repeatedly wipe 3-5 times.

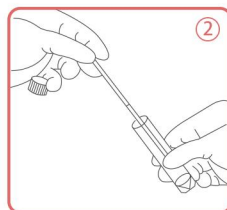

② Immerse the swab head after sampling preserved in liquid in the tube.

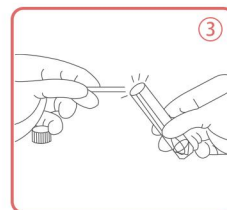

③ Break off the swab from the breaking point

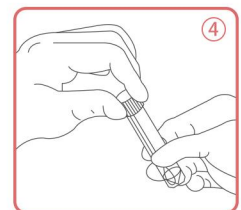

④ Tighten the lid and shake

